


## Easy-Ramp and Easy-Flex-Ramp

### Using the Vehicle Without a Passenger

After disembarking a passenger / unloading a wheelchair:

1. Fold the access ramp up until it audibly engages in the vertical position.
2. Raise the seat retainers.

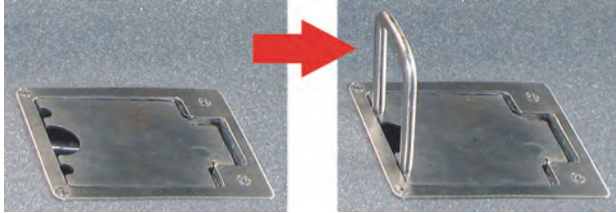

3. Unfold the vehicle's rear seats to the original position.
4. For Easy-Flex-Ramp: Push the operating lever to the right, to fold the access ramp away.



5. Fully fold the access ramp down inside the vehicle, until its bearing surfaces rest on the vehicle's floor at the sides of the floor section.

With the access ramp fully folded away and the rear seats in their original position, the vehicle can be used as a standard vehicle (only for Easy-Flex-Ramp).



**⚠ Attention! Max. permissible wheelchair access ramp load when deployed 300 kg.  
Max. permissible wheelchair access ramp load when folded away 100 kg.**

Issue: April 2017 / Subject to technical changes without notice.

AMF-Bruns

## BRIEF OPERATING INSTRUCTIONS

Dacia Dokker with Easy-Ramp and Easy-Flex-Ramp



**BRIEF OPERATING INSTRUCTIONS**  
Read thoroughly before use, keep readily available in the vehicle!

The brief operating instructions are not a replacement for the full Operating Instructions! Please read the full Operating Instructions thoroughly before using the product for the first time!



**AMF-Bruns GmbH & Co. KG**

Hauptstraße | 26689 Apen | Germany  
Telephone (Sales) +49 (0) 4489 / 72 72 -33 / -50  
Telephone (Service) +49 (0) 4489 / 72 72 -22 / -37  
Telefax +49 (0) 44 89 / 6245  
hubmatik@amf-bruns.de | [www.amf-bruns.de](http://www.amf-bruns.de)

Art.-No.: 10018973 | 04/2017



[www.amf-bruns.de](http://www.amf-bruns.de)



### Deploying the Access Ramp

1. Open the rear door / tailgate.
2. For Easy-Flex-Ramp Fold the access ramp up until it audibly engages in the vertical position.



3. Push the operating lever, to deploy the access ramp, to the left.




4. Fold the access ramp out and lower it to the ground.



**⚠ Caution! Keep hold of the access ramp until it is fully in contact with the ground. Allow the access ramp to deploy slowly.**

### Loading a Passenger / Wheelchair

1. Fold the vehicle's rear seats to the stowed position.
2. Fold the seat retainers down.

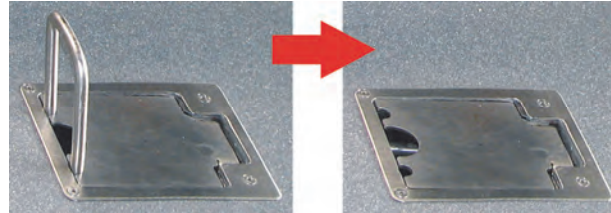

3. Accompany the wheelchair to a position in front of the access ramp.
4. Select the ON/OFF switch to "I".



5. Pull the electrically operated belts out to the wheelchair.
6. attach the electrically operated belts to the wheelchair.



7. Select the ON/OFF switch to "0".

**⚠ Warning! The wheelchair can run backwards out of control if the ON/OFF switch is selected to "I". The passenger in the wheelchair can suffer injury. Select the ON/OFF switch to "0". Push or drive the wheelchair up the access ramp only when this has been done.**

8. Push or drive the wheelchair into the vehicle.
9. Switch the motor of an electrically driven wheelchair OFF.

### Securing the Passenger and Wheelchair in the Vehicle

1. Attach the retractor belts to the wheelchair.
2. Clip the retractor belts onto the fittings in the rear area of the floor of the vehicle.



3. Tension the retractor belts by tightening the belt reels.
4. Apply the wheelchair's brake(s).
5. Secure the passenger in the wheelchair using the wheelchair and passenger restraint systems provided. Observe the relevant brief operating instructions when doing so.
6. Fold the access ramp up until it audibly engages in the vertical position.
7. Close the rear door / tailgate.

### Disembarking a Passenger / Unloading a Wheelchair

1. Open the rear door / tailgate.
2. Fold the access ramp out and slowly until it is in contact with the ground.
3. Release the retractor belt reels.
4. Disconnect the retractors from the fittings.
5. Detach the retractor belts from the wheelchair.
6. Switch the motor of an electrically driven wheelchair ON.
7. Release the wheelchair's brake(s).

**⚠ Warning! The wheelchair can roll backwards out of control. The passenger in the wheelchair can suffer injury. Stand behind the wheelchair to prevent it from rolling backwards. Only then, select the ON/OFF switch to "I".**

8. Select the ON/OFF switch to "I".
9. Accompany the wheelchair down from the access ramp.
10. Detach the electrically operated belts from the wheelchair.
11. Attach the electrically operated belts to the belt locks on both sides of the vehicle.
12. Select the ON/OFF switch to "0".
13. Fold the access ramp up until it audibly engages in the vertical position.
14. Close the rear door / tailgate